



FERNO[®]
When It's Critical[®]

SAERBAG³

SCOOPEXL[™]

Created for the rescue and transport of an injured patient in a difficult environment. It allows an immobilized patient to be transported on a Ferno **SCOPEXL**[™] stretcher, on a Ferno Millennia backboard or on a Ferno vacuum mattress and to be rescued using a winch and/or ropes in a difficult environment, thanks to its special integrated lifting bridle.

The rescue bag has been tested according EASA CS 29.865 a), amdt. III.

HIGHLY RESISTANT LAST GENERATION FABRIC ①

Highly resistant to traction, impermeable to 2,000 cm³ of water with a triple coating to avoid wear and tear over time. Water and oil repellent, stain and mold resistant treatment.

REINFORCED STITCHING

Reinforced stitching to ensure the bag works properly over time and is safe in critical situations.

The stitching of the integrated lifting bridle is reinforced with back tacking with bonded black thread.

THE COVER ②


The bag has a semi-rigid Mylar cover with 3 windows: one at the front and two at the sides. These allow the operator to monitor the patient and offer greater comfort during rescue operations. They offer the patient more space and 'breathability' in the face area, avoiding claustrophobia and increasing comfort during rescue operations. The large Mylar 'windows' also allow the operator to monitor the state of the patient and allow the patient to interact with the outside world, thus reducing anxiety. Mylar is highly resistant to traction, humidity and most chemical agents. It does not fray at temperatures from -70°C to +150°C. Wind resistant.

THE UNDERSIDE OF THE BAG ③

The underside of the bag has been designed for optimal resistance to tears and abrasion by being dragged across any rough or uneven surface. It is therefore suitable for all emergency situations and environments.

ADJUSTING THE LENGTH ④

The bag has an internal length adjustment system. This system allows the bag to be adapted to the patient immobilization device placed inside. It has two straps on both sides of the bag that can be easily adjusted by the operator.

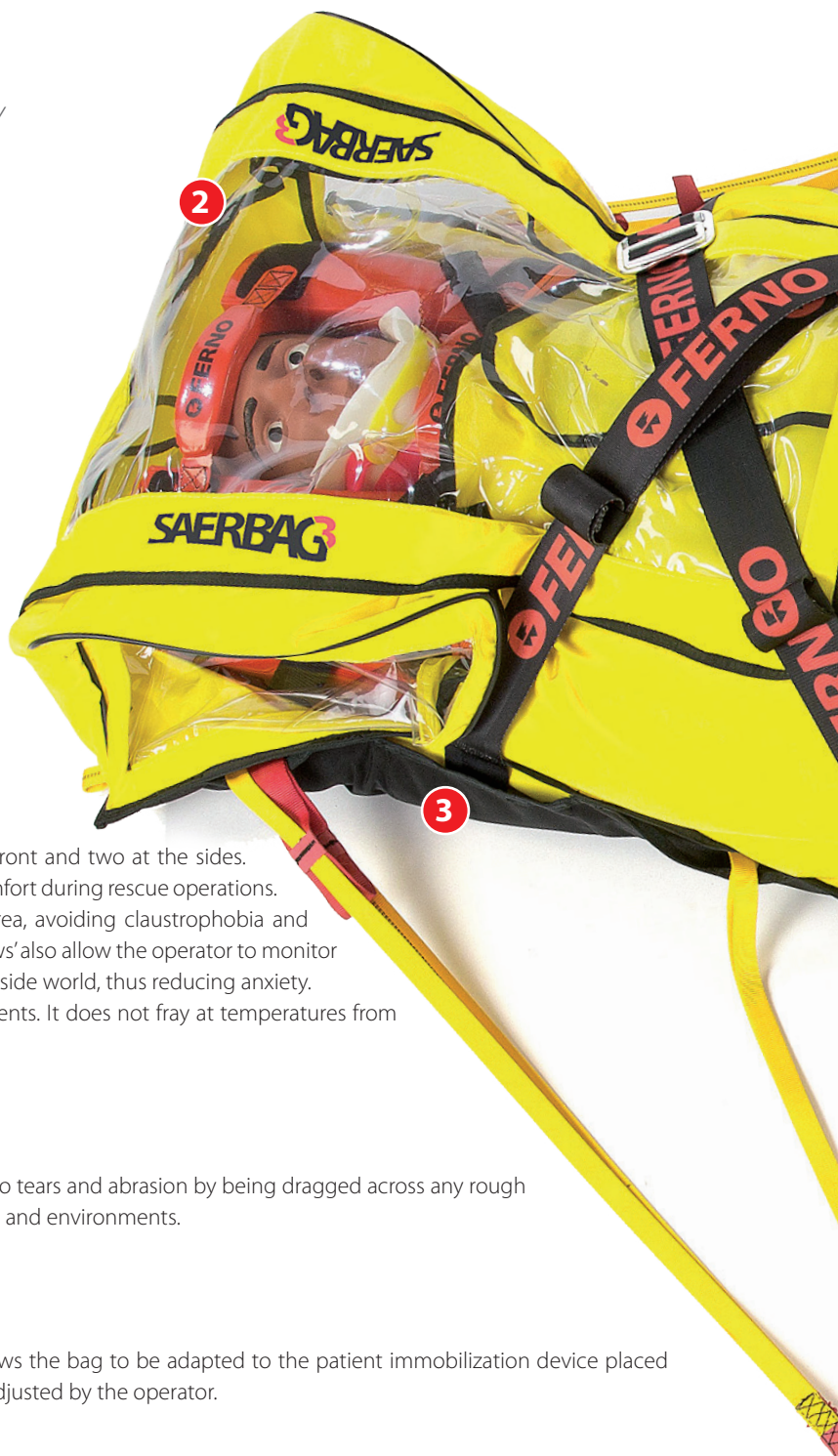

STRAPS

The 4 integrated straps can be easily adjusted to immobilize the patient correctly: 2 upper torso straps, 1 leg strap and 1 foot strap.





THE LIFTING BRIDLE ⁵

The integrated lifting bridle has 10 arms (5 per side) and two central lifting rings that karabiners are attached to for rescue operations. It has a shortening system with 4 integrated red loops.

LIFTING BRIDLE SHORTENING SYSTEM ⁶

The lifting bridle shortening system ensures a correct and adequate distribution of the load on the arms of the lifting bridle if the bag has been shortened to the length of the SCOPEXL stretcher inside.

The red loops of the lifting bridle shortening system are positioned on the outer arms of both sides of the lifting bridle.



8

EARS ANTI ROTATION SAIL ⁷

The bag has two EARS anti rotation sails, positioned on both sides to guarantee maximum compatibility with most means of transport with a rotating sail. The EARS anti rotation sails control rotations caused by the movement of the helicopter during rescue operations.

The sails have an orange handle outside the pocket.

Patented system.

ANTI ROTATION TAG-LINE LINKS ⁸

The bag has 6 anti rotation tag-line links at the head, side and foot area. Among the 6 links there are two top release systems.

TOP RELEASE SYSTEMS ⁹

These are positioned on both sides of the bag in the foot area. The two lateral top release systems allow the operator to undo the tag line at any time without a fuse system.

SECURING STRAPS ¹⁰

The 6 inner securing straps allow the SCOPEXL stretcher/backboard to be further secured inside the bag.

Each strap can be placed in 4 different positions, thanks to 4 loops sewn onto the bag. The operator can therefore easily and quickly decide where to position the straps according to the position of the handles in order to firmly secure the stretcher on the spot.

LIFTING HANDLES ¹¹

The bag has 6 high visibility red lifting handles at the head, center and foot area.





MONITOR CABLE ACCESS ¹²

The bag has monitor cable access that allows cables to be inserted into the device to monitor the patient.

STORAGE

The bag can be stored in the handy BAG SAERBAG III carrying case (supplied). The FLYING-BAG III bag/backpack is also available for transport. The Saerbag can be stored inside together with the SCOOPPEXL scoop stretcher so that the whole bag and stretcher system is always ready for use. The FLYING-BAG III can be carried on your shoulders.

TECHNICAL SPECIFICATIONS

Maximum length	2,000 mm	Weight	5 kg
Minimum length	1,450 mm	Load capacity	160 kg
Width	600 mm		

FLYING-BAG³

FLYING-BAG III BAG/BACKPACK

- This can be used in helicopter rescue operations thanks to a high visibility attachment point
- Can be carried on the shoulders. If the shoulder straps are not used they can be placed in their special pocket
- Hinged opening to insert the Saerbag III more easily
- Padded underside for greater resistance
- You can store the Saerbag III with the SCOOPPEXL stretcher thanks to the storage compartment for the SCOOPPEXL stretcher at the rear of the bag.



Ferno Italia Code	Description
SAERBAG III-Y	SAERBAG III yellow rescue bag for patient lifting and transport. Compatible with the SCOOPPEXL STRETCHER and backboards. With 2 anti rotation sails and patient cover. Transport and storage bag included.
KIT-FLYING III-Y	Kit for helicopter rescue. Includes: YELLOW SAERBAG III-Y with transport and storage bag, YELLOW SCOOPPEXL stretcher with pin and equipped with 4 straps mod. 773 and head immobilizer mod. 365-E (including transport/storage bag), 1 WizLoc cervical collar model 449-i, 1 single-patient high protection blanket MW1002 and FLYING-BAG III BAG/BACKPACK.
FLYING-BAG III	Bag/backpack specifically for KIT FLYING III, black
BAG SAERBAG III-Y	Yellow transport and storage bag for SAERBAG III
10-9900-005	Bag for transport and storage of head immobilizer 365-E
MW1002	Single-patient high protection blanket
EARS	Orange anti rotation sail for basket stretchers

Available in a military version on request.

The SAERBAG III is supplied with a maintenance handbook.



FERNO WASHINGTON ITALIA - Via B. Zallone n.26,
Pieve di Cento 40066 (BO) Italy - Tel. +39 051 6860028 / +39 051 6861508

