



COMPACT TREATMENT STATION

Unlike many predecessors, the FMS Compact Litter Stand can fold to approximately 1/2 its height. This simple feature allows our litter stand to be packed for mobile operations offering advanced treatment in forward areas.




FERNO Military Systems' Compact Treatment Station (CTS) is specifically designed with packability, mobility and immediate deployment in mind. The Compact Litter Stands fold small enough to fit into multiple deployment hard cases and various Load Out Bags. The compact size of the (CTS) provides US Forces with a readily available station for casualty treatment.

Optional transport case can store CTS Kit, SOF-Litter (quad-fold litters) and is compatible with current SOCOM TCCC CASEVAC sets. SOF NATO Stretcher sold separately.

P/N: FM840A128 **NSN:** 6530-01-618-7884 (does not include litter).

LITTER STANDS

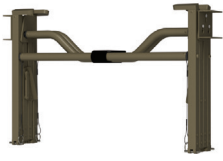

1 - Folded configuration (stowed)

Height	438 mm
Width	116 mm
Length	733 mm
Cubed	0.0368 cubic m
Weight	4.4 kg



2 - Pull legs apart to deploy

Height	427 mm
Width	330 mm
Length	733 mm
Cubed	0.103 cubic m
Weight	4.4 kg



3 - Deployed configuration (low pos.)

Height	424 mm
Width	1139 mm
Length	733 mm
Cubed	0.354 cubic m
Weight	4.4 kg



4 - Fully deployed configuration

Height	810 mm
Width	504 mm
Length	733 mm
Cubed	0.298 cubic m
Weight	4.4 kg

Extreme temperatures affect litter stand surface.
Use care for patient comfort.



Weight tolerance in fully deployed configuration (4).
Tested in pairs.

FMS Litter Stands are load and cycle tested in pairs and will support 500 lbs. FMS Compact Litter stands will accept all rigid NATO litters.

ACCESSORIES



IV POLE
FM840A104



BRACKET
FM840A126



MAYO TRAY KIT
FM840A127



MAYO TRAY
FM840A116

To Disinfect: Treat all surfaces with disinfectant following the disinfectant manufacturer's guidelines for application method and contact time.

To Clean: Hand-wash accessories with warm water and a mild detergent. Rinse with clean water. Dry with a towel or allow the board to air-dry.

Storage: Store the accessories only when dry.

Inspection: Regularly inspect each; at minimum, monthly.

DAVINCI KIT



Ferno Military's Compact Treatment Station DaVinci Kit (DVK) is compatible with our CTS accessory mounting points. The DVK provides rapidly installable extremity articulation platforms, allowing field medics to perform advanced field procedures or minor surgical procedures. Each Davinci CTS interface allows for an upper and lower medrail location.

FMS Litter Stands are compatible with most rigid NATO Litters.



Weight 10,8 kg per side

Length 8,3 cm

Height Variable

Depth 30,5 cm