## DIRECTIONS FOR USE

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**English:** Directions for use. **Germa Mini Tranz Mattress**  
**Svenska:** Bruksanvisning. **Germa Mini Tranz Mattress**  
**Dansk:** Brugsanvisning. **Germa Mini Tranz Madras**  
**Suomi:** Käyttöohje. **Germa Mini Tranz Mattress**  
**Norsk:** Brukerveiledning. **Germa Mini Tanz madras**

**Distributor:**

Ferno Norden AS  
Inlaget 298  
NO-3185 Skoppum  
Norway

Phone: +47 33 03 45 00

Fax: +47 33 03 45 10

E-mail: [fno@fernonorden.com](mailto:fno@fernonorden.com)

[www.fernonorden.com](http://www.fernonorden.com)

**Manufacturer:**

AB Germa  
Industrigatan 54-56  
SE-291 36 Kristianstad  
Sweden

Phone: +46 (0)44 12 30 30

Fax: +46 (0)44 10 31 79

E-mail: [info@germa.se](mailto:info@germa.se)

[www.germa.se](http://www.germa.se)

## English    Directions for use

### 1. Warning and caution statements

#### CAUTION

- Avoid handling the mattress next to sharp objects.
- Avoid handling the mattress next to open fire.
- On a current basis check that the mattress has no cracks.
- Own modifications on products not allowed – Your medical specialist to be responsible for any modification.
- When decided to take the product out of service and destroy – make sure to follow your local instructions or contact us for additional information.

Recommendation: Usable life of 3 years

### 2. Intended use

The **Germa Mini Tranz Mattress** is intended to be used as lifting and carry mattress to or on the stretcher during transportation. It is also a device intended to facilitate the transfer of the patient from one stretcher to another.

The mattress can be carried by two persons for a shorter distance, and a longer distance if two or more persons use additional carrying handles and belts. You can pull the mattress on the ground by the handles or by connecting a carrying belt or rope.

The mattress consists of an outer shell of airtight, fully welded, material that is filled with a thermofoam inlay and is fitted with handles on two or four sides for carrying or pulling.

### 3. Specifications

The Germa Mini Tranz mattress is compliant in accordance with SS EN 1865 and MDR 2017/745 Class 1

**Temperature range:** -30°C to +70°C

#### Parts/Materials

Outer shell: PVC coated fabric (flame retardant)  
Filling material: Thermo foam  
Plate in polypropene

Dimensions	Length (cm)	Width (cm)	Height (cm)	Weight (kg)	N <sup>o</sup> of handles
<b>Germa Mini Tranz Mattress</b>	136	77	2	2,7	10

### 4. Description

The Germa Mini Tranz Mattress is equipped with 4 handles at each side and with 2 handles at head end.

The mattress consists of an outer cover of hard-wearing PVC coated polyester weaved canvass with a foam filling.

### 5. Operating instructions

1. Place the injured patient in the medically desired position on the Germa Mini Tranz mattress
2. Lift and carry the mattress and patient carefully on to the stretcher.
3. Close the belts of the stretcher.
4. Check that the unit is stable and comfortable for the patient.
5. The injured patient is ready to be transported.
6. The Germa Mini Tranz Mattress can not be used as a stretcher mattress. Its dimensions allows it to fit on any commonly sold stretcher with mattress.
7. The Germa Mini Tranz Mattress can easily be slid over to another stretcher or trolley at the hospital or at change of transportation.

## 6. Cleaning and Disinfecting

At a regular basis, the mattress should be wiped with a moist cloth and dried, if dirty, the mattress may be washed with soapy water and rinsed with clean water.

Allow the mattress to dry before storage.

If the Mini Tranz Mattress needs disinfection first clean the mattress carefully and let it dry. Pour a solution containing 70 % alcohol or clean with most common disinfectants. Smear the transfer mattress with the cloth.

## 7. Repair of Mini Tranz Mattress

If damage occurs repair as follows:

- a) Find a patch in a suitable size and put glue on both the patch and the damaged spot on the transfer mattress.
- b) Wait for the glue to become dust dry (approx.2 min.) and place the patch on the damaged spot.
- c) Press firmly the whole area of the patch, so that there is a good contact and all air is pressed out under the patch.

The repair will receive full strength after 24 hours.

## 8. Storage

The Germa Mini Tranz Mattress is often positioned on top of the stretcher with a covering blanket.

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(To be assigned in by  
Manufacturer)

---

**Please print or type or fill in by MS Word, all sections a. – i. Fax or e-mail to your supplier. Please use repair order when reporting repairs not associated with complaints.**

---

---

For claims please specify :

Purchase Date

---

---

**Description of product**

---

---

**Description of Incident/Problem:** 1) What, 2) How?

---

---

---

When? In connection with: 1) transport, 2) unpacking, 3) preparing, 4) test, 5) cleaning, 6) normal use, 7) training, 8) disassembly, 9) assembly, 10) storage, 11) other:

---

---

**Info on the product, prior to problem,**

---

Storage: 1) stock, 2) site of use, 3) packed, 4) unpacked, 5) softpack, 6) other:

---

Used cleaning and disinfecting agents/Sterilizing method:

---

Pattern of use: 1) stand-by for emergency, 2) routine use, 3) stand by, 4) other:

---

Environment: 1) Indoor, 2) Outdoor, 3) Heat, 4) Cold, 5) Humidity, 6) Sunlight, 7) Gases, 8) other:

---

---

Which **actions** have been: a) taken, b) planned by the reporter or user at the site of incident?

1) examination of the product on site, 2) replacement of the product, 3) other:

---

Suggested actions by the distributor/complainant

---

---

**Returning of samples/faulty product(s),**

- 1) Already shipped/Will be shipped to \_\_\_\_\_,
- 2) Pending request from manufacturer,
- 3) Product/parts not available:

---

Date:

---

**We acknowledge the receipt of this report by manufacturer and will start processing it immediately. Please use the assigned complaint report number for further reference.**

**Your contact person in this matter is:** \_\_\_\_\_

Name:

Direct Tel.

Direct Fax.

Init/Date:

---