


Stanag 2040 NATO Uni/Duo Fold Stretcher MANUAL



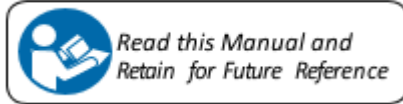

Disclaimer

This manual contains general instructions for the use, operation and care of this stretcher. The instructions are not all-inclusive. Safe and proper use of this stretcher is solely at the discretion of the user. Safety information is included as a service to the user. All other safety measures taken by the user should be within and under consideration of applicable regulations. It is recommended that training on the proper use of this stretcher be provided before using this stretcher in an actual situation. Retain this manual for future reference. Include it with the stretcher in the event of transfer to new users. Additional free copies are available upon request from FERNO Manufacturing.

Model STANAG 2040 Uni Fold Stretcher Product Specifications

Length with extended handgrips2290 mm
Length with retracted handgrips1988 mm
Length of vinyl cover 1915 mm
Width of open stretcher 570 mm
Width of closed stretcher 92 mm
Height - open 157 mm
Diameter of supporting round tubes-	
Aluminium alloy 6082-T6 36/32 mm
Diameter of handgrips 31 mm
Weight 6 Kg
Maximum load capacity200 Kg

TIP: Regular cleaning helps reduce the risk of transmitting disease and enables the equipment to function at its optimum

WARNING

Untrained operators can cause injury or be injured. Permit only trained personnel to operate the stretcher.

Improper use of the stretcher can cause injury. Use the stretcher only for the purpose described in this manual.

Improper operation can cause injury. Operate the stretcher only as described in this manual.

An unattended patient can be injured. Stay with the patient at all times.

An unrestrained patient can fall off the stretcher and be injured. Use restraints to secure the patient on the stretcher.

An improperly-secured stretcher can separate from the transporter and cause injury. Attach the stretcher securely and verify that the slam latches have locked.

Helpers can cause injury. Maintain control of the stretcher, operate the controls and direct all helpers.

Helpers can be injured. Show helpers where to grasp the stretcher to avoid pinch points.

Improper maintenance can cause injury. Maintain the stretcher only as described in this manual.

Improper parts and service can cause injury. Use only Ferno parts and Ferno-approved service on the stretcher.

Modifying the stretcher can cause injury and damage. Use the stretcher only as designed by Ferno.

Model STANAG 2040 Duo Fold Stretcher Product Specifications

Length with extended handgrips2290 mm
Length with retracted handgrips1988 mm
Length folded 937 mm
Length of vinyl cover 1915 mm
Width of open stretcher 570 mm
Width of closed stretcher 99 mm
Height - open 157 mm
Diameter of supporting round tubes-	
Aluminium alloy 6082-T6 36/32 mm
Diameter of handgrips31 mm
Weight 8,7 Kg
Maximum load capacity 200 Kg

Product Number	Product Name	GTIN/UDI
60-0102-032	STANAG 2040 Duo Fold NATO Stretcher	8588008129630
60-0102-034	STANAG 2040 Uni Fold NATO Stretcher	8588008129647

STANAG 2040 NATO Uni/Duo Fold Stretcher Instructions

[Product Name] Medical Device STANAG 2040 NATO Uni/Duo Fold Stretcher

[Intended Use]

The Model STANAG 2040 NATO Uni/Duo Fold Stretcher is emergency patient-handling device. Each stretcher in the series is for professional use by a minimum of two trained operators. The stretcher can be carried by the operators.

In this manual, the generic term "stretcher" is used when the feature is shared by all models in the series. The individual stretcher name is used when features differ.

[Application]


The Model STANAG 2040 NATO Uni/Duo Fold Stretcher is the emergency stretcher of choice for: Military Medics, Athletic Trainers, First Responders, EMS, Wilderness Rescue, Ski Patrols, Outdoor Adventurers, Safety Engineers, Veterinarians and more.

[Directions for use]

Unfolding and folding the stretchers



Model STANAG 2040 NATO Duo Fold Stretcher

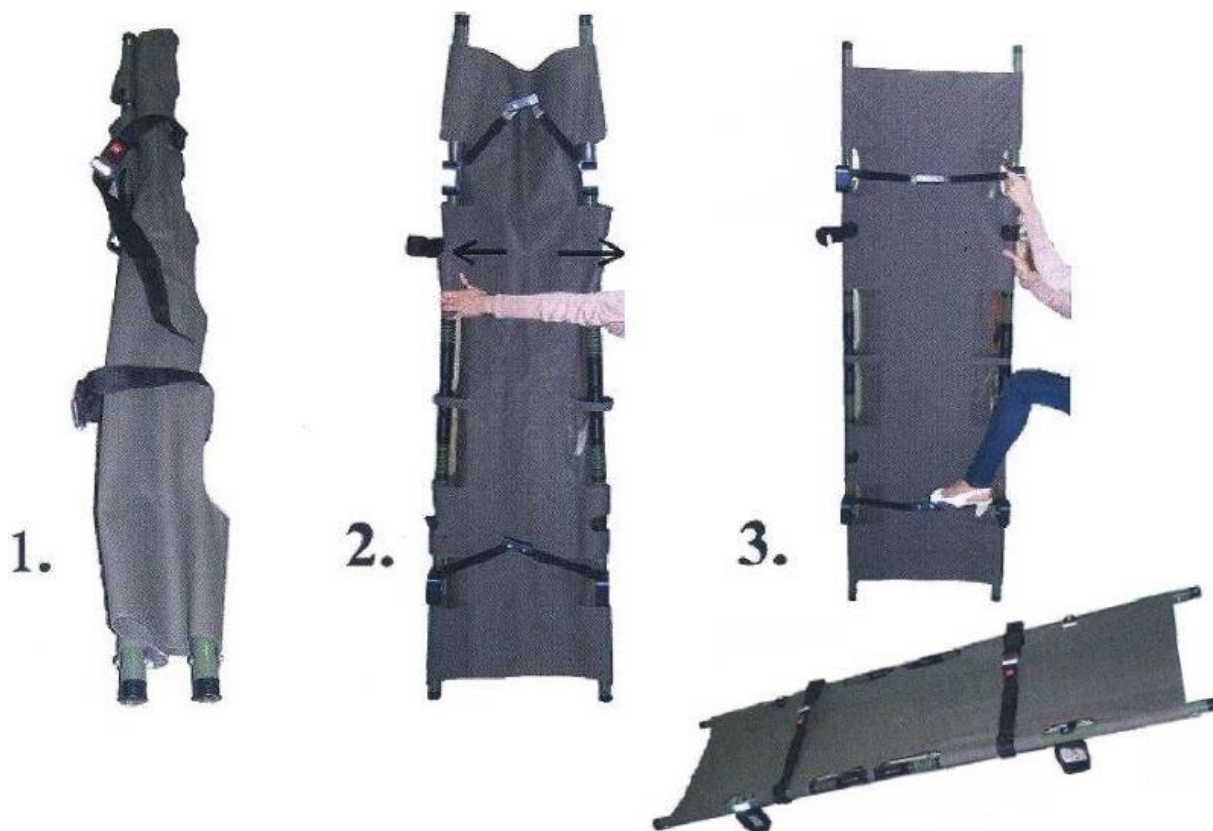

[Cleaning and Disinfection]

Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time. Ferno recommends you inspect the stretcher for damage as you disinfect it.

[Inspecting the stretcher]

Have your service's equipment maintenance personnel inspect the stretcher regularly. If inspection shows damage or excessive wear, remove the stretcher from service until repair is made.

[Parts and services]

To order Ferno parts and for professional stretcher repair, contact FERNO or your Ferno distributor. Your distributor is the only agent authorized by Ferno to manage, service and repair Ferno products.

Ferno Slovakia Contact:

Ferno Slovakia s.r.o. Bošáca 893
913 07 Slovakia
Tel: + 421 (0) 32 7708 010
Fax: + 421 (0) 32 7708 011

[Storage]

Avoid High temperature & moisture

[Declaration of Conformity]

The Models STANAG 2040 NATO Uni/Duo Fold Stretcher mentioned in this Manual were designed, manufactured and tested to meet the specification of EN 1789 and EN 1865 standards.



Please use under the guidance of the doctor or nurse



Ferno Washingtom Italia S.r.l.
via B.Zallone, n26, 40066
Pieve di Cento,
Bologna, Italy
+39.051.6860028