



User instructions

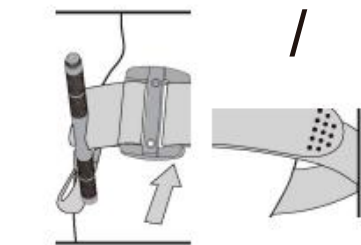


ONE-HAND APPLICATION

Thread the strap into a loop over the injured extremity and position the tourniquet 5-Scm above the bleeding directly against the skin.

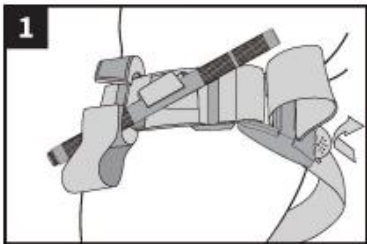


TWO-HANDED APPLICATION


Place the strap around the injured limb and thread the end of the strap through the plastic buckle. Position the tourniquet 5-Scm above the bleeding directly against the skin.

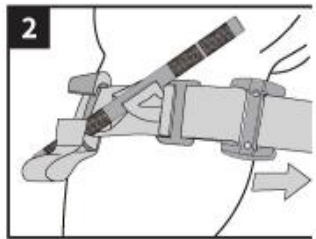


The strap should always pass through the buckle and its serrated edge. Tighten the strap firmly and fasten it to itself all the way around the limb but not over the crotch where the TIME label is located. The strap should be tightened so tightly that 3 fingers cannot be pushed under it. If it is too loose, it must be tightened and reattached.

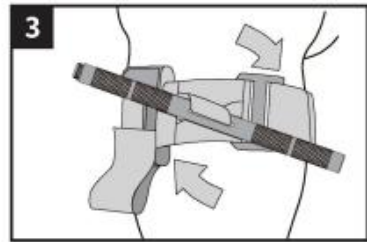

Turn the tourniquet until the bleeding is stopped. Also check that there is no pulse below the tourniquet. The distal pulse should cease when it is correctly applied.

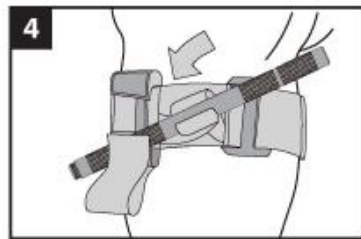


Securely tighten the tourniquet in the cruciate ligament so that it cannot come loose. Check that bleeding and distal pulse have stopped. Consider tightening further or applying a second tourniquet above the first. Evaluate the results.

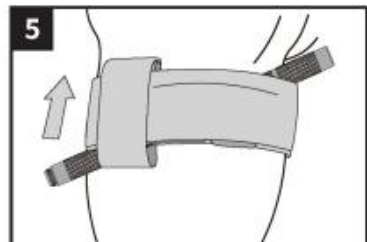

Place the strap through the crook and over the tensioning pin. Secure the strap and tensioning pin by pulling the TIME label around the strap. Mark the time of application with a pen in the tensioning pin. Consider whether additional securing of the dressing with tape is required. Ensure the tourniquet is visible so that it cannot be overlooked by healthcare professionals.

FÖRVARING AV TOURNIQUET I QUICK LAUNCH KONFIGURATION

STORAGE OF THE TOURNIQUET I QUICK LAUNCH CONFIGURATION

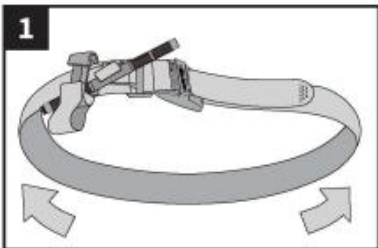


Thread the end of the strap through the buckle. Pull through about 20cm of the strap, fold it back and attach it to itself.



Flatten the loop formed by the strap. Place the buckle in the center of the flattened strap.




Fold the tourniquet in half so that the buckle is at one end. Make sure the tab marked TIME is free and ready to use as shown in the picture.

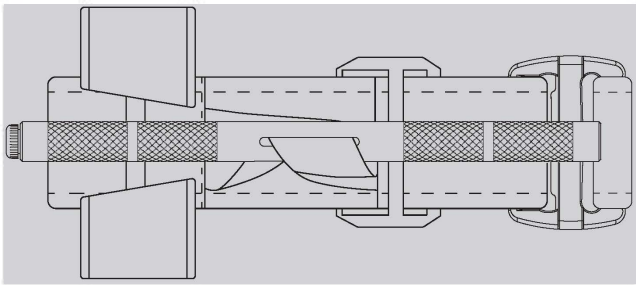

SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel, The Netherlands
Tel: +31(0)10 3034500; +31(0)2021 11106 E-mail: ec.rep@sungogroup.com



WUXI EMSRUN TECHNOLOGY CO.,LTD
No.100 Fengbin Road, Wuxi City, Jiangsu Province, 214000, P.R.Of China.

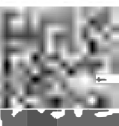

(01) 06974567740331
(11) 241115
(10) 241115
(17) 291114

[LOT] 20241115

[DATE] 20241115

[EXPIRATION] 20291114



Lot Number



Date of Manufacture



Expiration date



Manufacture



Authorized representative in the European Community/European Union



Do not re-use



Medical device

Manufactured for/Importer: ViTri Medical AB, Sweden. www.vitri.se



Only to be used by persons with appropriate training.
För användning endast av personer med lämplig utbildning.